

# RESMED

## S9 Escape™

POSITIVE AIRWAY PRESSURE DEVICE



## Welcome Guide

English

**S9**™ Series  
More. Comfort.

**S9**<sup>™</sup> | Series  
More. Comfort.

# Welcome

Thank you for choosing the S9 Escape. Before operating this device, please read the entire Welcome and Information Guides.




## S9 at a glance

The S9 system comprises the following elements:

- S9 device • SlimLine air tubing • 90W or 30W power supply unit • S9 travel bag • SD card
- S9 SD card protective folder.

Optional components include:

- H5i heated humidifier • H5i water tub • Standard air tubing • ClimateLine heated tubing
- 90W power supply unit • 30W power supply unit (does not support H5i).

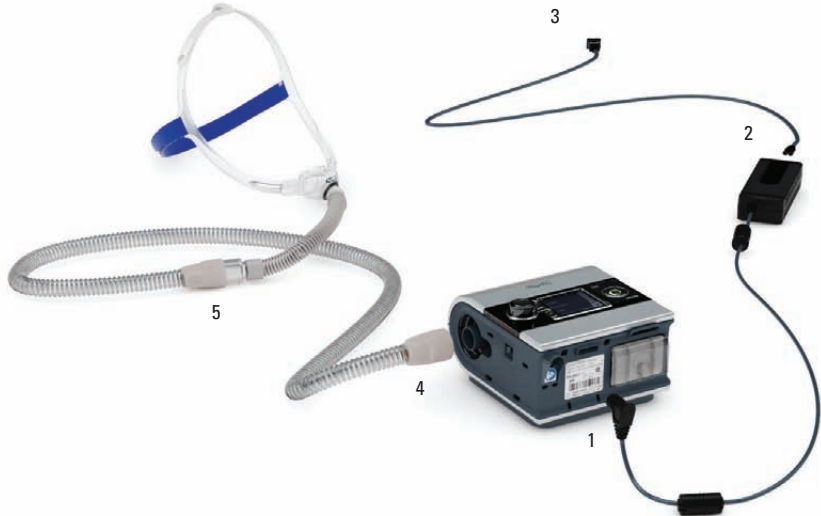

## Setup

1. Connect the DC plug of the power supply unit to the rear of the S9.
2. Connect the power cord to the power supply unit.
3. Plug the other end of the power cord into the power outlet.
4. Connect one end of the air tubing firmly onto the air outlet.
5. Connect the assembled mask system to the free end of the air tubing.

### Notes:

- For more information on assembling your mask see your mask user guide.
- Recommended masks are available on [www.resmed.com](http://www.resmed.com) on the Products page under Service & Support.



**Start/Stop button**  
Starts or stops  
treatment.

**Home menu**

**LCD screen**

**Escape**

**EPR**


**Info menu**  
Allows you to  
view your sleep  
statistics.

**Setup menu**  
Allows you to  
make changes to  
settings.


**Push dial**  
Turning the dial  
allows you to  
scroll through the  
menu and change  
settings. Pushing the  
dial allows you to  
enter into a menu and  
confirm your choice.


**Key:**


Home 


Humidity  
level 


Ramp 


Heated tube 

Climate  
Control 

Start/Stop 

Push dial 

Setup menu 

Info menu 


## S9 basics

The control panel of your S9 device includes an LCD screen as well as a Start/Stop button, Push dial and the Info and Setup menu keys.


## Navigating the menus

In general, to navigate the menus:




1. Turn  until the parameter you require is displayed in blue.




2. Press . The selection is highlighted in orange.





3. Turn  until you see the setting that you require.



4. Press  to confirm your choice. The screen returns to blue.



### Getting started

1. Make sure the power is connected.
2. Adjust the ramp time if required.
3. Fit your mask as described in your mask user guide.
4. To start therapy press .
5. Lie down and arrange the air tubing so that it is free to move if you turn in your sleep.
6. To stop treatment at any time press .

## Cleaning and maintenance

Refer to your mask and humidifier user guides for detailed instructions regarding the care of this device.

### Daily

Remove the air tubing by pulling on the finger grips on the cuff. Hang it in a clean, dry place until next use.

### Weekly

1. Remove the air tubing from the S9 device and the mask.
2. Wash the air tubing in warm water using mild detergent.
3. Rinse thoroughly, hang, and allow to dry.
4. Before next use, reconnect the air tubing to the air outlet and mask.


### Monthly

1. Wipe the exterior of the S9 with a damp cloth and mild detergent.
2. Check the air filter for holes and blockage by dirt or dust.

### Replacing the air filter

Replace the air filter every six months (or more often if necessary).

1. Remove the air filter cover from the back of the S9 device.
2. Remove and discard the old air filter.
3. Insert a new ResMed air filter ensuring that it is sitting flat in the air filter cover.
4. Replace the air filter cover.

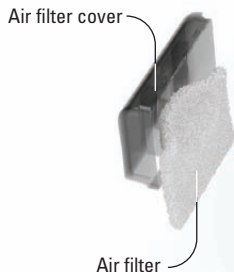


## SD card

An SD card has been supplied to gather therapy data from your S9 device and provide settings updates from your clinician. When instructed to do so, disconnect the S9 device from the power outlet, remove your SD card, insert it in the protective folder and send it to your clinician.

### Removing the card

1. Push in the SD card to release it.
2. Remove the card.
3. Insert the card into the protective folder.
4. Send the protective folder back to your clinician as instructed.

For more information on removing and inserting your card refer to the S9 SD Card Protective Folder provided with your device. Please retain the S9 SD Card Protective Folder for future use.

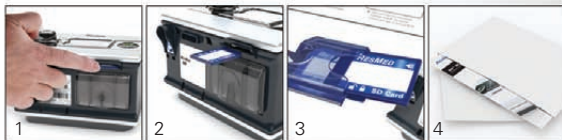

### Notes:

- For Troubleshooting information refer to the Information Guide.
- For more product information see [www.resmed.com](http://www.resmed.com).

**S9**<sup>™</sup> | Series  
More. Comfort.

368347Eng/1 2010-10

S9 Escape

WELCOME

EUR 1/3

**Manufacturer:** ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. **Distributed by:** ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See [www.resmed.com](http://www.resmed.com) for other ResMed locations worldwide. S9: Protected by patents: AU 691200, AU 697652, AU 702820, AU 709279, AU 724589, AU 730844, AU 736723, AU 750095, AU 750761, AU 764761, AU 779327, EP 0651971, EP 0661071, EP 0920845, EP 0927538, EP 0934723, EP 1028769, EP 1126893, EP 1144036, EP 1502618, JP 3580776, JP 3778797, NZ 504595, US 5199424, US 5245995, US 5522382, US 5704345, US 6029665, US 6138675, US 6240921, US 6363933, US 6367474, US 6425395, US 6502572, US 6591834, US 6675797, US 6745768, US 6817361, US 6988498, US 7040317, US 7100608, US 7320320, US 7537010. Other patents pending. Protected by design registrations: AU 329025, JP 1377401, JP 1377402, JP 1377403, NZ 411762, NZ 413090. Other designs pending. S9 is a trademark of ResMed Ltd and is registered in U.S. Patent and Trademark Office. © 2010 ResMed Ltd.

