# RESMED

## Tubing Wrap

STANDARD AND SLIMLINE

### User Instructions

English • Français • Deutsch • Italiano • Español • Português • Nederlands • Svenska • Dansk • Norsk • Suomi  
• 简体中文 • 繁體中文

#### English

Tubing wraps are intended to reduce condensation and temperature loss in ResMed air tubing. There are two tubing wraps:

- **Standard** for use with standard ResMed air tubing 6' (1.8 m) or 6' 6" (2 m) long, 0.7" (19 mm) inner diameter.
- **SlimLine** for use with SlimLine air tubing and ClimateLine heated air tubing 6' (1.8 m) long, 0.6" (15 mm) inner diameter.

### Fitting instructions

#### Standard 6' (1.8 m) Tubing Only

To fit the tubing wrap to the Standard 6' (1.8 m) tubing, the tubing wrap must be shortened. The free end of the tubing wrap has two Velcro® tabs on its internal surface.

To shorten the tubing wrap, fold the end of the wrap so the two velcro tabs are fastened together. You may now fit the tubing wrap as described below.

#### Standard (6'/1.8 m and 6' 6"/2 m) and SlimLine Tubing

1. Fully unzip the tubing wrap and unfasten both Velcro straps.
2. Open the tubing wrap out and lay it down with the inside facing up.
3. Pass one end of the air tubing through the closed end of the tubing wrap until the rubber cuff is poking out.
4. Keeping the air tubing cuff in position, zip the tubing wrap completely around the air tubing.
5. Fasten both of the Velcro straps around the tubing wrap.

Ensure that the tag end of the zip (all tubes) and the zipper pull tab (6' tubes only) are secured under the straps.

#### WARNING

Do not use the tubing wrap if the room temperature is above 86°F (30°C). At higher temperatures the air temperature at the mask may be too hot.

### Cleaning instructions

The tubing wrap is made from nylon and polyester. It is latex free.

The tubing wrap should be hand washed or dry cleaned only. Wash the tubing wrap separately when new, as the colour may run. Do not bleach.

To wash the tubing wrap:

1. Wash in cold to warm water using a mild washing solution.
2. Rinse well and drip dry.

**Note:** Hand-washing your tubing wrap does not replace the need to clean your air tubing.

### Symbols

The following symbols may appear on your tubing wrap or packaging.

-  Caution;  Follow instructions for use;
-  Manufacturer;  Latex free.

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**Manufacturer:** ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia **Distributed by:** ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See [www.resmed.com](http://www.resmed.com) for other ResMed locations worldwide. Patents and design registrations pending.

Velcro is a registered trademark of Velcro Industries B.V.

©2010 ResMed Ltd. 368241/1 10 04